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
RPS 300

Integrated lift with CAN BUS managing system

Original instruction translation

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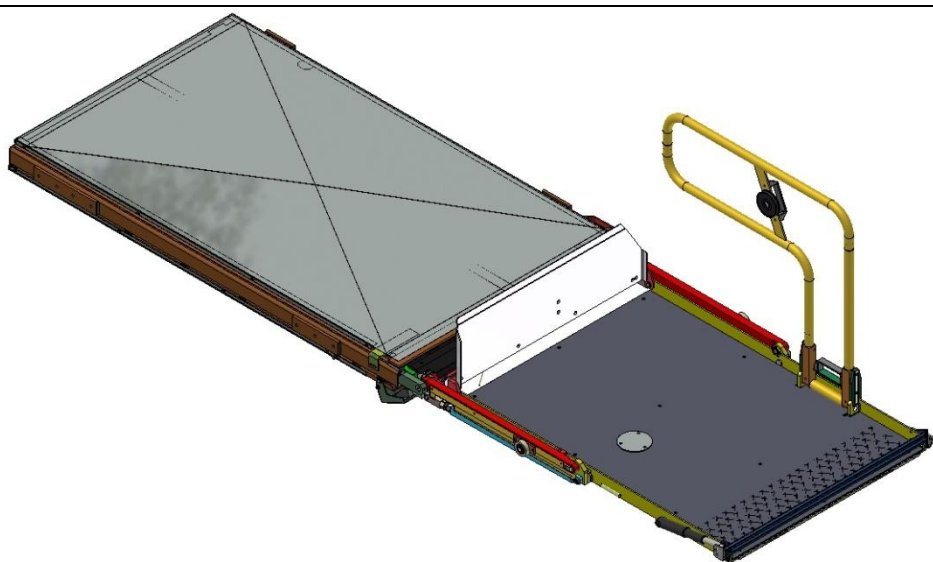English text of this document is the translation of the original manufacturer instructions. If further assistance is required, please refer to the original document or forward a request to **Mobility Networks Ltd**

No part of this manual may be reproduced, stored by a filing system or transmitted to third parties in any form or by any means without the prior written permission of **Mobility Networks Ltd**.

Mobility Networks Ltd reserves the right to carry out product changes and improvements in order to enhance their quality, at any time and without notice, event if such modification this will not reported in this publication.

The Testing and Inspection operations of this product must be performed by authorized **Mobility Networks Ltd** centres.

Keep this manual in the vehicle for future reference.

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1. LOG VERIFICATION FOR THE FIRST COMMISSIONING

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Compilare i campi come descritto al Cap. 3 del manuale di Installazione e Service
 Compile the fields as described to the Chapter 3 of the Fitting and Service handbook.

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(trascrivere i dati relativi al sollevatore in oggetto riportati in tabella 2)
 (return the data of the lift, this document refers to: data are listed in table 2):

SOLLEVATORE LIFT	RPS 300						
PORTATA CAPABILITY	BRACCI ARMS	G_1	d_1	G_2	d_2	P	d_p
[kg]	[mm]	[daN]	[mm]	[daN]	[mm]	[daN]	[mm]
	_____	_____	_____	_____	_____	_____	_____

INSTALLAZIONE LATERALE SIDE INSTALLATION ☐			
C [mm]		B_{min} [mm]	
_____		_____	
B [mm]	R_{min} [daN]		T [daN]
_____	_____	≤	_____

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Name and address of the person authorized to compile the technical file:

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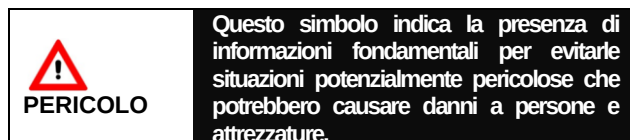
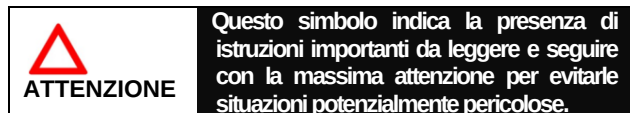
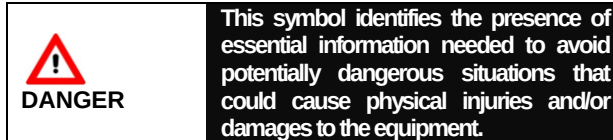
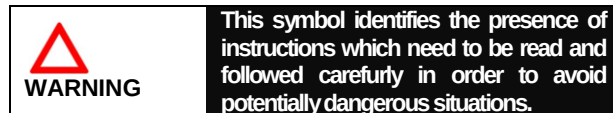
SAFETY RULES AND SYMBOLS

Some precaution are listed in this manual in order to identify potentially dangerous situation in advance and the relevant instructions to avoid them, identified by the following symbols:

To ensure smooth and safe operation, it is necessary to know the exact procedure for the installation and service of the wheelchair lift.

WARNING

Use the lift for the wheelchairs by faithfully following the instructions in this manual, if the instructions in this manual are not fully understood or there is the need for additional information, please contact Mobility Networks Ltd immediately. These instructions non-fulfilment can cause serious damage to the equipment and to the vehicle as well as serious injuries to people.

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INDEX

1. LOG VERIFICATION FOR THE FIRST COMMISSIONING	3
2 DECLARATION OF CONFORMITY	2
3 PREFACE	9
3.1 CLASSIFICATION AND GENERAL CONSIDERATIONS	9
3.2 DOCUMENTATION	9
3.3 WARRANTY	9
3.4 TECHNICAL DESCRIPTION	10
3.5 TECHICAL SPECIFICATIONS	11
4 STANDARDS AND SAFETY DEVICES	12
4.1 SAFETY REGULATIONS	12
4.2 SAFETY DEVICES	16
4.3 PLATES AND SIGNAL STICKERS FITTING	17
5. CONTROLS	18
6 USE AND OPERATION	21
6.1 PRE-OPERATION CHECKS	21
6.2 SWITCH ON THE LIFT	22
6.3 LIFTING STEP OPERATION	22
6.4 MANOUVERING GOING OUT	22
6.5 LOWERING	23
6.6 LIFTING	23
6.7 LIFT STOWING	24
6.8 EXPECTED SPECIAL CONDITIONS OF USE	25
6.9 CONTROL BEFORE VEHICLE MOVEMENTS	25
6.10 EMERGENCY OPERATING SYSTEM	26
6.10.1 GOING OUT AND LOWERING	26
6.10.2 LIFTING	27
6.10.3 REENTRY	27
6.10.4 ELECTRONIC POWER UNIT RESET	28
7 MAINTENANCE	29
7.1 PERIODIC VISUAL CHECKS	29
7.2 CLENGING OF THE LOADING PLATFORM	31
7.3 ROUTINE INSPECTIONS	32
7.4 SPECIAL INSPECTION	43

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3 PREFACE

3.1 Classification and general considerations

The wheelchair lift “**RPS 300**”, manufactured by **Mobility Networks Ltd**, which is the object of this instruction manual, is in compliance with the safety requirements of 2006/42/CE directive. The “**RPS 300**” wheelchair lift in keeping with the aforementioned directive, is to be considered as a ‘*machine*’ and therefore a declaration of conformity has been released, according to annex II of the 2006/42/CE directive.

All the necessary instructions for the correct installation of the wheelchair lift are detailed for the installer together with a list of maintenance instructions for servicing procedures.

3.2 Documentation

The user should receive with the “**RPS 300**” wheelchair lift:

- the original copy of the manufacturer's declaration of conformity;
- the installation conformity declaration both filled and signed by the installer;
- use and maintenance manual filled by the installer.
- spare parts catalogue (on request).

Mobility Networks Ltd is at your complete disposal for further clarification and instructions.

3.3 Warranty

Mobility Networks Ltd assures the wheelchair lift for 24 months from the date of delivery. The warranty covers for defects concerning material quality and product manufacturing.

The warranty does not cover consumables and defects or failures as a result of installation procedure incorrectly carried out or not in compliance with the manufacturer's instructions.

The warranty becomes void in case of impacts caused by accidents and/or tampering carried out by personnel not authorized by **Mobility Networks Ltd**.

Mobility Networks Ltd refuses all responsibility for damages caused by:

- improper use of the electro-hydraulic lift;
- platform overloading (more than 300kg);
- a failure in carrying out “use and maintenance” manual instructions;
- a failure in carrying out maintenance operation as detailed in the “use and maintenance” manual;
- interventions or modifications to the lift without **Mobility Networks Ltd** authorization;
- lift safety devices removed or tampering;
- wrong manoeuvres when moving the hydraulic lift;
- presence of objects or people in lift working area.

WARNING

Manufacture's regular inspections dates non-fulfilment will automatically make void the warranty.

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3.4 Technical description

RPS 300 wheelchair lift (Fig. 1) has been exclusively designed for lifting and lowering disabled people and/or only one operator onto/from the vehicle floor to which it is fixed.

Electro-hydraulic lift **RPS 300** is mainly composed by:

- main aluminium frame (cassette);
- steel sliding carriage, with arms and lifting cylinders connected to it, that allows platform to exit from the structure;
- loading platform made of aluminium extruded profiles with antiskid surface;
- electro-hydraulic power unit, with electro-pump, oil tank, electric valves and emergency manual command.

A side installation are normally preferred.

On an horizontal level, platform and cassette are moved by an electric gear-motor (each) with rack gear system; on a vertical level, it is moved by hydraulic cylinders with parallelogram kinematics. Lift is fed by vehicle batteries by means of a spanner switch and it is managed by an electronic power unit.

The manual remote control is provided with push-buttons allowing platform enter/exit and lifting/lowering.

In emergency conditions, platform can be controlled not only by remote control, but also by pump and valve manuals and pull locking handle, even if electric installation failure has occurred, the lift can be used in safety conditions.

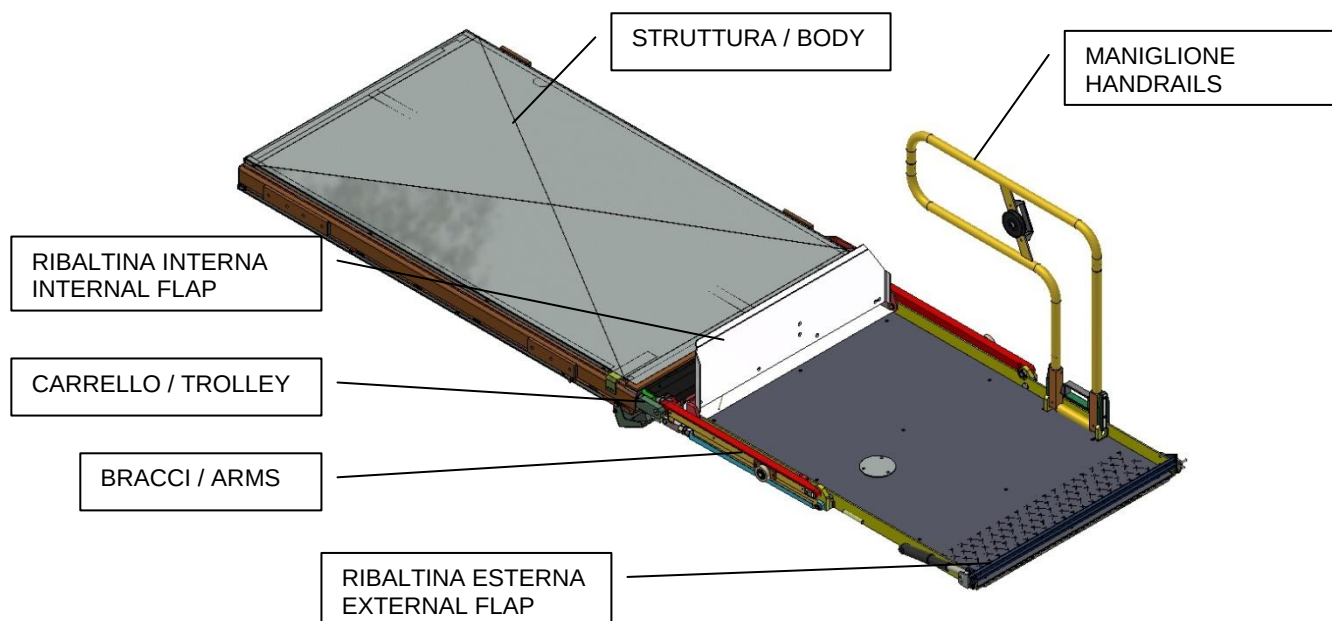
Fig. 1

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3.5 Technical specifications

Find following the main technical specifications of **RPS 300** wheelchair lift:

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4 STANDARDS AND SAFETY DEVICES

4.1 Safety Regulations

Below are a set of rules which, will ensure that no harm is done to people, animals or things whilst using the wheelchair lift:

- The lift covered by this manual, should only be used for the lifting / lowering of disabled people in wheelchairs onto / from the vehicle where it is installed; any other use is a misuse and could cause damage to people, animals and things;
- the use of the lift is permitted exclusively to authorized personnel who are perfectly familiar with the location and the function of all the controls, lights, indicators and safety standards;
- do not use the lift if you are not sure you have understood all the manoeuvring procedures;
- failure to observe the safety rules and the instructions in this manual may result in accidents to the passenger on the wheelchair, to the operator, to the vehicle and to third parties;
- before operating the lift, carefully read all the labels and the safety stickers applied to it;
- do not wear dangling or ragged clothing (ties, scarves, loose shirts) and jewellery (watches, necklaces, bracelets) that could get caught in the moving or stationary parts of the machine;
- do not make any changes which may alter the structure of the lift/hoist that could compromise the operational security. If necessary, contact the authorized **Mobility Networks Ltd** centre first for an optimal resolution of the problem;
- in case of tampering or unauthorized modifications, **Mobility Networks Ltd** assumes no liability and the warranty and CE certification is deemed as lapsed;
- do not tamper with the calibration and safety devices of the lift / hoist;
- the use of the lift is forbidden:
 - in areas at risk of fire,
 - in the presence of corrosives, explosives or dust harmful to the health of the operators;
 - inside magnetic fields;
- the lift / hoist should be installed on the vehicles approved for use on public roads. Therefore, before entering into circulation, make sure that the vehicle is in compliance with the existing legislation;
- before loading the lift / hoist, make sure that the weight of the disabled passengers in addition to the wheelchair and possibly the operator, does not exceed the maximum permitted load (300kg);

PRIOR TO OPERATIONS

- always make sure that the work area is adequately illuminated, otherwise arrange for its proper illumination;

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Do not use the lift in breakdown.

The lift is to be used at an environmental temperature of MIN -15°C and MAX 48°C in order to prevent malfunctions and damage to the system; in cold weather (up to max -40°C), operate the hydraulic system without load for a few minutes, so as to circulate and heat the oil.

Park outside the traffic flow on a flat and level surface and make sure there is enough space for manoeuvring the lift.

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DANGER

Do not try to locate the oil leaks with your hands or other body parts: a leakage, even if very small, can pierce the skin causing painful sores and infections that require prompt medical intervention.

DURING THE OPERATION

- verify that the radius of action of the lift is always free of obstructions;
- do not stop and do not let people stop in the radius of action of the lift;
- the operator who operates the lift must always keep a safe distance and stay in a position that allows him to have control over the entire radius of action of the lift;
- all the manoeuvres should be done with maximum caution, keeping the head, body and limbs away from the areas of shearing, crushing, entrapping and impact;
- check that, during the movements, the platform does not interfere with any part of the vehicle, also make sure that the connecting cables of the wired remote control do not get caught in the corners of the vehicle and / or parts of the lift;
- when operating the lift do not place any objects on the push button panel and other commands;
- it is forbidden to insert objects between the moving parts of the lift;
- it is forbidden to throw objects to and from the lift;
- do not lift or to low from the loading platform of the lift when it is moving;
- the brakes should be compulsorily applied to the wheelchair before starting the manoeuvre of lifting / lowering;
- do not move the lift in operational sequences of command in contrast with those shown in Chapter 6 of this manual;

DANGER

Do not start the vehicle engine in enclosed areas, unless they are equipped with an effective ventilation system and a system for disposal of exhaust gases.

WARNING

The lift has been designed exclusively for lifting and lowering disabled persons in wheelchairs with a single operator in charge to / from the floor of the car to which it is fixed; the use of the lift for loading /unloading other persons and/or objects is therefore forbidden.

DANGER

The operator is required to stop the machine if the people in the radius of action of the same, do not go away despite his requests.

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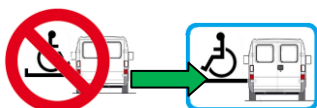
DANGER

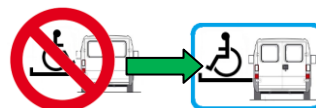
It is prohibited to exceed the nominal load; also ensure that the disabled person and any operator are always in perfect balance on the platform and that the platform never exceeds the inclination of $\pm 10^\circ$ as regards the horizontal plane.

DANGER

Stop any manoeuvre if there is a risk of the disabled person and/or operator falling down.

WARNING

The disabled person must always be positioned with his back to the vehicle.

WARNING

By withdrawing the load, the loading capacity halves at a distance.

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AT THE END OF OPERATION

- before closing the lift, make sure that no object was forgotten on the loading platform;
- disable the system and then disconnect and store the remote control inside the passenger compartment of the vehicle.

DANGER

Before moving the vehicle check that the lift is securely closed and the indicator light in the cabin is off.

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4.2 Safety devices

The wheelchairs lift "RPS 300" is equipped with devices of protection and safety:

- **Hydraulic installation general maximum pressure valve:**
operating in order to reduce the hydraulic oil pressure to values not higher than the ones of the working pressure (required for lifting the maximum load allowed).
- **25 A fuse on the electronic power unit:**
in order to protect the control unit on hydraulic power unit.
- **100 A fuse on the battery:**
located in the positive cable of the battery and before of the enable-key for all the protection power line.
- **External flap to stop and connect platform to the ground and Internal flap to stop and connect platform to vehicle loading deck:**
to make wheelchair going up/down the platform easier; they are provided with torsion springs in order to keep them vertical when platform is far from ground or vehicle loading deck position.
- **Emergency hand pump:**
allows to use the lift in safety conditions in case of electric installation failure.
- **Gear-motor disengagement:**
for emergency manual manoeuvres.
- **Power supply general disconnecting switch (battery isolator system).**
- **Reflecting strips bounding platform overall dimensions.**
- **Antiskid lift surface.**
- **Intermittent lights of platform overall dimensions.**

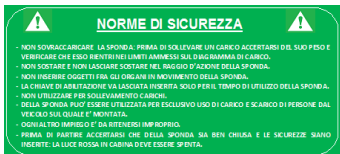
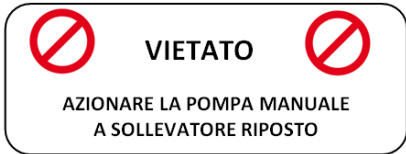
4.3. Targhette identificative e adesivi

4.3 Plates and signal stickers fitting

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An identification tag and safety stickers have been applied to the lift, as follows:

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 **WARNING** In the case of loss or excessive wear, identification tags and safety stickers should be rapidly replaced. Please contact an authorized Mobility Networks Ltd centre for details.

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5. CONTROLS

5.1 Remote control

1. Plug remote control
2. LED ON
3. DEPLOY/LIFTING button
4. LOWERING button
5. STOWING button

The remote control (**Fig.3**) allows the operator to maneuver the lift at a safe distance from above the loading platform.

In order to avoid unintentional movements, all the controls are "with automatic return" to neutral position.

In order to avoid unauthorized manoeuvres in absence of the operator and to disconnect the control panel from the electric power supply given by the batteries of the vehicle, it is necessary to remove the command-enable key from the commutator.

To activate the keypad you must:

- Insert the keypad plug in the proper electronic control box socket "1" (Fig.3);
- Screw the security screws of the plug to guarantee the block and to avoid that the keypad accidentally disconnects itself during the operation;
- rotate the power supply red key in clockwise direction (Fig.6): the light will turn on "2"(Fig.3).

WARNING Never leave the lift unattended with the enabling switch system switched to the ON position.

WARNING Commands, plaiyed by different controls at the sometime, are not accepted by lift electronic and control unit.

WARNING It is important that the user of the lift has full vision lift warning area.

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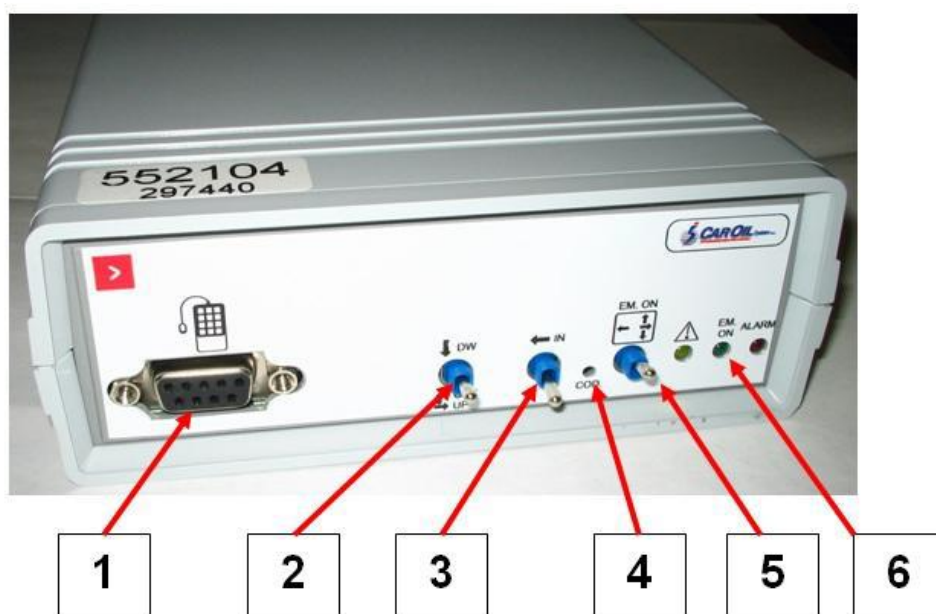
5.2 Electronic control box

Electronic control box (under) is normally located in the vehicle internal part near lift exit door, or inside the vehicle boot sheltered from water and dampness.

Selectors "2" and "3" are used only in case of emergency, when happens a keypad failure.

DANGER - Do not leave the lift unattended with the control enable-key inserted in the control panel switch.

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- 1) Socket for push-button panel remote control panel;
- 2) "EXIT/LIFTING" and "LOWERING" emergency selector;
- 3) "RETRACTING" emergency selector;
- 4) "RESET" button;
- 5) Controls selector from control panel (emergency) or from push-button panel;
- 6) Working power pack light.

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5.3 Hydraulic unit

- 1) Contactor
- 2) Electro-pump electric engine
- 3) Oil tank
- 4) Adjustable throttle
- 5) Lowering electro-valve
- 6) Hand pump
- 7) Oil tank cap
- 8) Enable key
- 9) Max pressure valve
- 10) Electronic power CAN BUS

WARNING

When the vehicle engine is switched off, it is suggested not to operate the power pack for more than one minute at a time, in order to not overload batteries.

In order to assure to the electro-pump "2" (Fig. 4) an optimally flow of hydraulic oil, it is necessary that the tank "3" (Fig. 4) is always full of oil.

It is possible to check the hydraulic oil level into the tank by unscrewing and removing the cap 7" (Fig. 4) with the lift stowed. If it is necessary to fill up oil "TEXACO RANDO HDZ 15", control the level oil in the tank is above the "MIN" mark and under the "MAX" mark. At the end of the operation, screw the cap.

WARNING

When filling up oil, do not mix oils having different features and gradations.

WARNING

When checking and filling up oil, keep platform on STOWED.

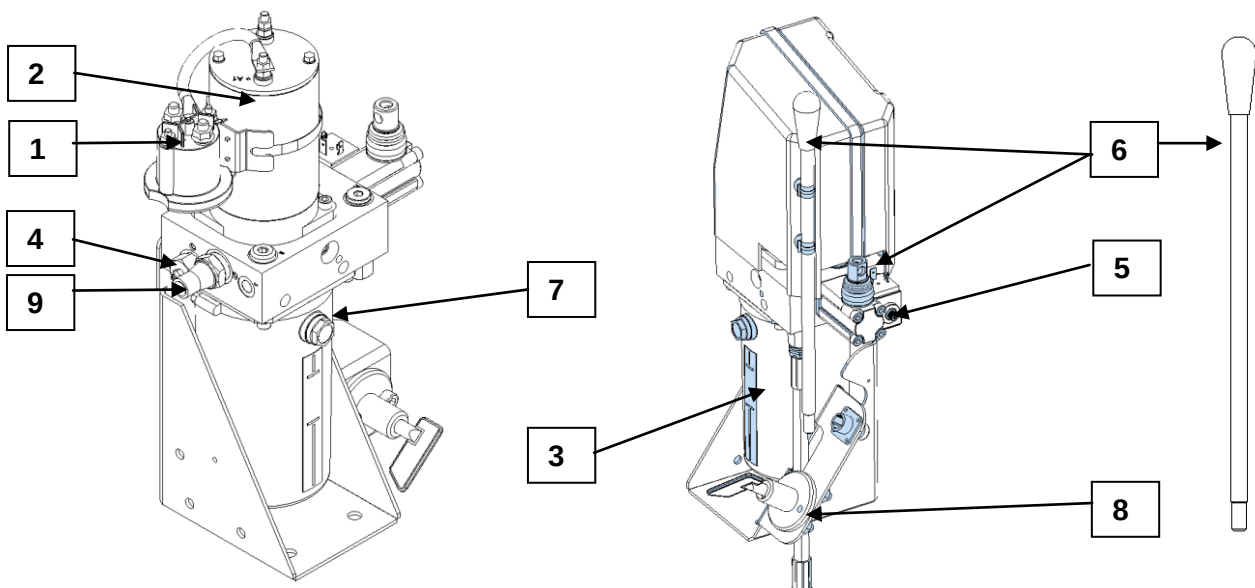
Fig. 4

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6 USE AND OPERATION

DANGER

Before starting using the lift, you must properly brake the vehicle by activating the parking brake and, if necessary, you must use wheel clamp wedges.

WARNING

Ensure that the vehicle is parked on a flat surface, horizontal and free of clutter.

6.1 Pre-operation checks

Before using the wheelchair lift it is mandatory to perform the following steps:

- ensure that there are no missing or damaged parts of the machine and structural failure;
- check that the safety labels and stickers are visible and legible;
- check vehicle batteries charge level;
- check hydraulic oil level contained in the tank and, if necessary, fill it up with oil having the same features. These operations must be carried out with lift stowed;
- always keep the machine moving elements clean from dirt;
- check that all pins are correctly housed in their seats, and that they are not lacking of their respective locking stops;
- check the integrity of the spiral cable of the wired remote control and the electrical connectors;
- check for leaks of hydraulic oil;
- check the fasteners for all the pins of the lifting arms;
- ensure that there are no loose screws.

In case of any kind of failure, do not use the lift and contact an authorized Mobility Networks Ltd centre.

DANGER

Do not start the vehicle engine up in closed areas if they are not provided with efficient ventilation and exhaust gas disposal systems.

WARNING

Check jacks efficiency: when stopping the control, the platform must stop.

WARNING

Make sure that the relative movement of the platform corresponds to each command without jamming and unusual noises.

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Fig.5

DANGER

If there are movements of the platform that are different from those provided for that particular command, do not use the lift and contact an authorized Mobility Networks Ltd centre.

6.2. Switch on the lift

In order to carry out this operation, rotate the disconnecting switch red spanner in clockwise direction (you must hear a trigger clearly): thus, you feed all the unit and the red warning light, located on the dashboard, must switch on (Fig.5).

6.3. Lifting step operation

WARNING

If you carry out this operation from the door under which the lift is installed, it is necessary to close it and then open it again in order to carry out the lifting step operation.

As a consequence, if you open the door, the platform exits to lock in the position allowing lifting step operation.

The safety cap opening and the loading platform exit occur simultaneously.

6.4. Manouvering going out

Keep the remote control (Fig. 6) and connect it to the plug on control unit box; therefore, press the push-button "3" for to outgoing platform;

- keep the deploying push-button "3" for to lifting the platform;
- Lift the handle, in vertical position, by locking it with special tightening.

WARNING

Once the disabled person has lifted on the platform or before starting the lifting or lowering operation, the wheelchair MUST BE braked.


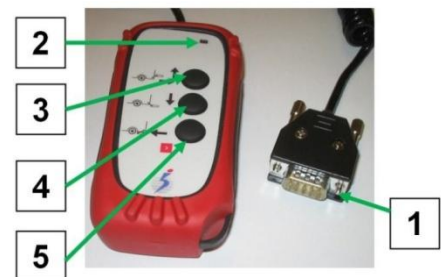
Fig. 6

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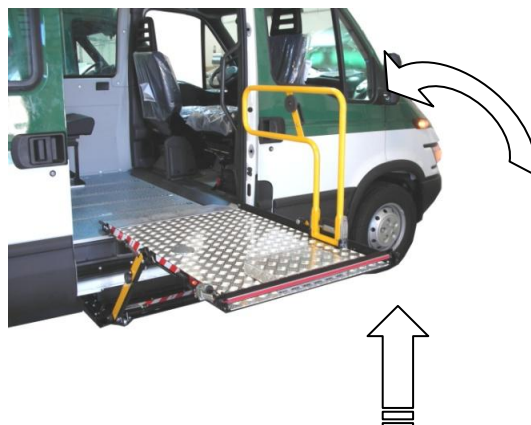6.5 Lowering

To carry out the operation, comply with the following instructions:

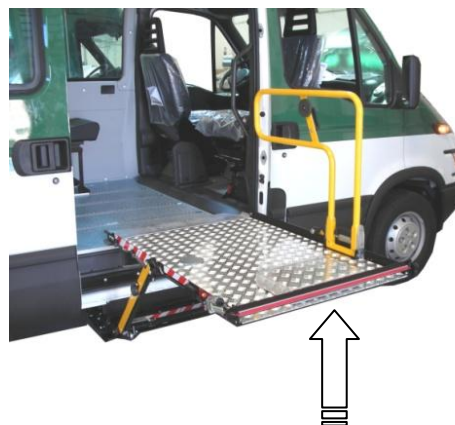
- During lowering operation, the platform is always on the horizontal plane and shifts from the loading level with an arch trajectory. Once having reached the ground, the platform can be loaded.
- Keep the lowering push-button pressed "4" (Fig. 6) as far as obtaining the automatic ramp platform flap on the ground.

6.6 Lifting

The platform is always on the horizontal plane and shifts from the ground to the vehicle loading level with an arch trajectory.

- Once having put the disabled person up on the platform, with shoulders turned towards the vehicle, and having braked the wheelchair, press the lifting push-button "3" (Fig. 6) as far as the platform reaches the loading level.
- Once having reached the vehicle loading level, the internal flap touch on vehicle level and the platform will stop automatically.

⚠ WARNING: check that the internal flap overcome the load floor of 20÷30 mm.

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Fig. 7

6.7 Lift stowing

After having checked that no objects were left on the lift, carry out the closing operation by complying with the following instructions:

- Unlock the handle, in vertical position, with special tightening, and place it parallel to the platform.
- Push the button "5" (*Fig. 6*) for total reentry .

The closure of internal flap is automatically.

If you close the door under which the lift is installed, the platform entries completely.

The safety cap closing and the platform entry occur simultaneously.

WARNING

Before closing operation, make sure not to leave any object on the platform surface and remove any possible dirt.

At the end of the operations:

Rotate the key (*Fig. 7*) IN THE SWITCH in anticlockwise direction as far as the GREEN warning light, located on control unit (*fig.6*).

Check that the warning light indicating "platform pulling out" is switched off.

Remove the key from the switch and keep it into vehicle cabin.

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6.8 Expected special conditions of use

They are extreme temperatures, such as $-40^{\circ}\text{C} \div -80^{\circ}$ and/or extreme environmental conditions (dusty, muddy, damp areas at high altitude, etc.) forcing the machine to operate in a bearing way.

In any case, it is important to check:

- electric components keep insulation features unchanged as well as good conditions of cables cleaning and flexibility;
- hydraulic installation components and jacks keep sealing features and hydraulic jacks capacity as well as the flexibility of rubber pipelines;
- mechanical components are always well-protected from rust;
- hydraulic oil level into the tank is always at an optimal level;
- electrolyte level and charge vehicle battery are always at an optimal level;
- hydraulic oil viscosity complies with climate, in order not to have oil too fluid in a warm climates and oil too thick in a cold climates.

6.9 Control before vehicle movements

- Make sure that the platform is correctly closed.

WARNING

Before starting the vehicle verify that warning light on electronic control box is turned off and remote control is stowed in the cockpit.

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6.10 Emergency operating system

Emergency manual controls are to be used to completely move the lift in case of lift failure, or in case of lack of current supply.

Emergency manual controls must be exclusively used to help the disabled passenger during lifting/lowering onto/from the vehicle and to close the opened lift, allowing to the vehicle to start again; then contact immediately **Mobility Networks Ltd** authorized service centre.

We are going to describe a complete cycle of the lift use (exit, lifting, lowering and entry) when it is exclusively moved by means of emergency manual controls.

6.10.1 Going out and lowering

Pull the red handle on right side of the lift (Fig.9),

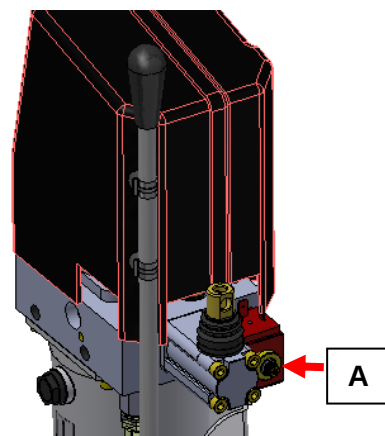
Fig. 11

Open the door under which the lift is installed.

Open the lift cap and keep the lift external edges with two hands, then pull the platform towards outside in order to open it completely (Fig. 10). **Lock the platform end running by means of locking red handle.**

To lower platform to the ground, turn counterclockwise lowering valve on the hydraulic unit (Fig.11).

Once having reached the ground in order to open the flap, it is necessary to remove the lock spit-pin and the pin, then it is possible to lower the flap on the ground with hands. **Mounted the lock spit-pin for closed the external flap.**

WARNING

Turn clockwise the lowering valve (Fig.11) for give the hydraulic pression of the lift. If you do not do this operation the platform does not lifting.

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6.10.2 Lifting

⚠ Make sure that the lowering valve (A) is closed first the lower monoevring.

⚠ Before starting to pump, check that the platform is completely out and opened, in order to avoid causing accidents.

- Pull out the pump hand staff "P" (Fig. 12) from its support "B".
- Pump as far as the platform reaches the loading level and the internal flap is joined to it;
- Verify that internal flap is on the floor and it is higher maximum 20÷30 mm than it.
- Lift the handle, in vertical position, by locking it with special tightening.

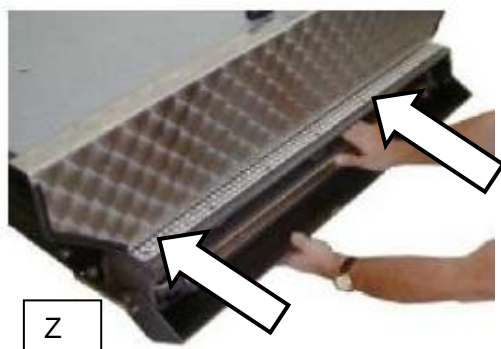
6.10.3 Reentry

- In case platform is load floor: rotate the lowering valve A (Fig.11) in anticlockwise direction for to lower the platform until plane of reentry;
- In case platform is a ground floor: is necessary pump with leave manual pump B as far as the platform reaches plane of reentry.
- take with both hands the external edge of the platform and push it towards the insider part of the vehicle, so that it retracts completely (Z);

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Fig. 13

- Lock platform again by pushing the button above unlocking handle (Fig. 13) and close the door;
- put the hand pump lever in its seat.

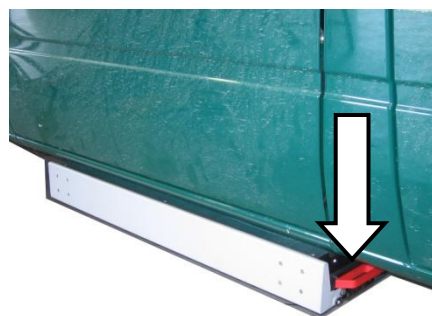
Fig. 14

⚠ WARNING When the platform entry operation is almost ended, take care to the cap closing, which occurs flexibly by means of springs: it is necessary to take the cap with the hand during its running. Crushing danger.

6.10.4 Electronic power unit reset

If system loses operation references (when the electronic power unit and all the platform are powered), reset the electronic power unit

To carry out the electronic power unit reset, carry out the following operations:

- close the lift manually (as it is explained in the paragraph 6.10): the electronic power unit "zero" is set to read the "all inside" signal, at the departure;
- Push the internal "RESET" button through "A" hole (Fig.13) for 1-2 seconds;
- release "RESET" button;
- check the movements of the lift.

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7 MAINTENANCE

Proper maintenance allows you to work in complete safety and ensures longevity of the wheelchair lift “RPS 300”.

User is therefore obliged to follow the instructions described in the following pages, strictly respecting the relevant deadlines.

WARNING

User is allowed to carry out only controls listed and the operation listed in *Chapters.7.1* and *7.2*.

WARNING

In order to ensure functionality and safety of the lift and maintain the warranty valid, user is obliged to carry out routine inspection in an authorized Mobility Networks Ltd centre, strictly respecting the relevant deadlines.

WARNING

In case of anomalies and malfunctions of the lift, user is obliged to carry out special inspection visits in an authorized Mobility Networks Ltd centre.

WARNING

Do not allow unauthorized personnel to carry out interventions on the lift in any manner.

WARNING

After 2000 HOURS of work and anyway after 10 YEARS of service, lift must undergo an inspection and a comprehensive review by an authorized Mobility Networks Ltd centre which will issue a new manual with a new assessment of the first commissioning.

OTHERWISE, THE LIFT MUST BE REMOVED FROM SERVICE.

7.1 Periodic visual checks

Find following visual checks and the relevant frequencies at which user must perform them in order to ensure the complete efficiency of the lift.

BEFORE EACH USE CHECK:

- the integrity of the spiral cable of the wired remote control;
- there are no missing or damaged parts of the machine;
- electrical cables and pipes run freely;
- there are no hardened parts, jams, unusual noises or bad contacts.

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EVERY WEEK CHECK:

- vehicle battery is charged;
- control oil level (Chapter 5.2).

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WARNING

When checking and filling up oil, keep lift stowed.

EVERY MONTH CHECK:

- there are no oxidation points in the structure;
- there are no loose screws;
- the safety label and stickers are visible and legible;
- lift functions correctly when empty, by performing a complete cycle of deploying, lowering to the ground, ascent and closure.

WARNING

If there are anomalies or malfunctions, do not use the lift and contact an authorized Mobility Networks Ltd centre.

WARNING

Special inspection interventions should be performed only by authorized Mobility Networks Ltd centres which will record the action in the appropriate form in Cap.7.4.

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7.2 Clening of the loading platform

The loading platform of the lift should be kept clean from dirt, mud and any other deposit in order to preserve the anti-skid characteristics.

WARNING

For cleaning the platform do not use jets of water at high pressure but degreasing agents.

WARNING

The oily rags should not be crowded together because in addition to being pollutants, they represent a major risk of fire. Therefore place them in appropriate closed containers and dispose them as required by the ruling law.

WARNING

The degreasing products should be disposed in accordance with anti-pollution regulations. Do not pour these substances on the ground, in drainage pipes, sewers and waterways.

WARNING

During the manoeuvres on the platform always use shoes with non-slip soles.

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7.3 Routine inspections

Compliance with the deadlines of ordinary inspections allows you to work in complete safety and ensures greater longevity to the "RPS 300" lift, in addition to maintaining the warranty valid until maturity.

The user is obliged to 6 month inspect the lift in an authorized **Mobility Networks Ltd** centre, which will perform the checks listed below and any relevant interventions necessary and will compile the related forms attached here below.

WARNING

When returning the vehicle verify that the Service Centre has compiled and signed the form attached below, related to the ordinary inspection just executed.

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Mobility Networks Holdings Ltd
Mobility Networks is a trading name of Mobility Networks (Holdings) Ltd.