

REFER TO THE FULL MANUAL available at mobilitynetworksgroup.com

STANDARD OPERATION

In a safe area, park the vehicle on level ground, make sure there is enough room around the vehicle to enable safe operation.

Open door(s) and secure to keep them open.


BEFORE OPERATING THE LIFT WITH THE APP:

Make sure the lift and App are paired.

The PIN can be found on the Power Pack LCD Screen

POWER ON lift (A) - Operator MUST stand clear of lift



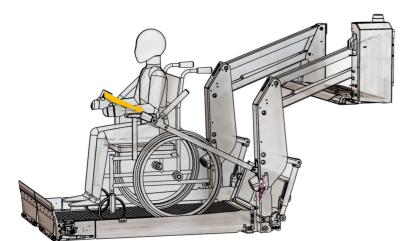

Loading Platform:

The lift can now be loaded.

Push and hold DOWN button until platform stops at the horizontal loading position on the ground.

Release button.

Once stopped at ground level the platform is ready to be loaded or unloaded.

When using the lift always fit Handrail Seatbelt

Raising Platform:

Push and hold UP button until platform stops at the horizontal loading position on the floor of the vehicle.





After making sure that no object has been left on the loading platform, proceed as follows for the closing manoeuvre:

Push and hold STOW button until platform stops at the loading position on the floor of the vehicle. Keep pressing until the power is interrupted.





EMERGENCY OPERATION



Emergency manual controls are to be used to operate the lift in case of power supply failure. They are not intended for 'normal' use as an alternative to powered operation.

Emergency manual controls must be exclusively used to help the user during lifting/lowering onto/off the vehicle and to close the opened lift. This will allow the vehicle to start again.

DEPLOY/DOWN

On the hydraulic unit, turn the manual override valve counterclockwise (Max 1/4 turn) lowering using the black knob (marked by its relevant label).

The platform will open then descend until contact is made with the ground or if the lowering valve is closed (by turning the knob clockwise).




Before any other operation, close the lowering valve, by turning its knob clockwise.

UP/STOW

Tighten the lever, into the threaded hole. (inside the vertical slot marked by the relevant label)

Operate the pump manually alternating up / down vertical movements.

The platform will reach vehicle floor level to allow user on / off the platform. Continue to operate the pump to stow the platform.

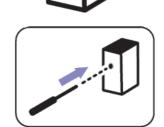





MAINTENANCE

Regular Maintenance and inspection is required and records kept, see manual for more details

Service Interval	Service Type
Daily	Daily
2 Weeks	2 Weeks
1000 cycles (or 3 months - whichever first)	Α
2000 cycles (or 6 months - whichever first)	В
3000 cycles (or 9 months - whichever first)	Α
4000 cycles (or 12 months - whichever first)	С
8000 cycles (or 24 months - whichever first)	D

WARRANTY may be void without these.

CLEANING

All cleaning should be carried out by hand using a gentle cloth or sponge soaked in non-aggressive detergent and then rinsed with a cloth dampened with water. Ensure all moving parts are relubricated where necessary, see below.

LUBRICATION

Grease all pins, use ACF-50 or silicon spray for all other parts.

